

OREGON HEALTH & SCIENCE UNIVERSITY
Hospitals and Clinics
Point of Care

Piccolo Maintenance Procedure

Weekly or As Needed:

1. Clean the exterior of the analyzer
 - a. Clean with a soft cloth dampened with a mild detergent or cleaning solution, or a 10% bleach solution, or a 30% isopropyl alcohol solution.
 - b. Do not spray or pour detergents or other solutions directly onto the analyzer.
 - c. Wipe the entire surface of the analyzer, including the keypad.
2. Clean the display screen
 - a. Clean by using a soft, lint-free cloth dampened with a glass-cleaning fluid.
Do not use any cleaner containing alcohol

Every 6 Months:

1. Remove the air filter:
 - a. Power down the analyzer and unplug the power cord from the back panel.
 - b. Grasp the black mesh filter in the circular opening and pull the filter free.
2. Wash the filter:
 - a. Use warm soapy water.
 - b. Dry the filter thoroughly before replacing it in the analyzer.
3. Replace the filter:
 - a. Tuck the sides in behind the edges of the circular opening.
 - b. Replace the rear panel.
4. Power up the analyzer:
 - a. Reconnect the analyzer to the power supply and plug it in.
 - b. The analyzer returns to standby mode.
 - c. If the analyzer reports an error, repeat the installation procedure.
 - d. If the error persists, contact Abaxis Technical Support at 1-800-822-2947.

As Needed:

3. Changing printer paper roll
 - a. Take new roll out of package.
 - b. Push plastic cover (on top-back of instrument) open.
 - c. Insert roll ensuring the lead edge is facing toward you on the bottom.
 - d. Close cover.

Software Updates:

1. When a new software update arrives, POCT will install.
 - a. From Home screen, press the **Settings** icon.
 - b. Press **More Settings**
 - c. Press **Software Update**.
 - d. Place CD in the drawer, press the drawer inward to close it. Press **Continue**.

Revised 6/13/11