

Burning DICOM images from DynaCAD

** CDs hold 700mb and DVDs hold 4700mb. Breast MRI cases range from about 600mb up to 900mb, depending on the number of slices and sequences.

The DynaCAD will burn DICOM information to a DVD

- * Insert blank DVD into the "RW" drive
- * Open the patient browser
- * Open the patient to be burned and select the series you want on the DVD
- * Right click on these series and choose "Create DICOM CD"
- * Select "Series Level" when window opens
- * DynaCAD will prepare this data for burning, so wait.....
- * The burn window will open
- * Name the DVD
- * In the drop down box, choose the "RW" drive
- * Click on BURN
- * A status bar will appear and once the bar is gone, the DVD is completed