



Let's get started.

WWW.PIXEL.LABCORP.COM

Kit includes:



1 Saline
collection tube



1 Pack of
cotton swabs




1 Gel pack



1 Biohazard
specimen bag



1 Shipping box

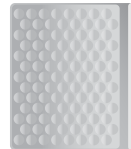

1 Insulated foil
pouch



FedEx return pack
UN 3373

Intended use: to collect a nasal swab for purpose of in vitro diagnostic testing in a CLIA laboratory.
Rx only

IVD

Distributed by Pixel by LabCorp
531 South Spring Street
Burlington, NC 27215

PXC42420-4

Self-Collection kit instructions

For In Vitro Diagnostic Use

REGISTER ONLINE



1

When you are ready to collect your sample, register your kit online at www.pixel.labcorp.com/register and type in the 12-digit barcode located on your collection tube.

2




Visit <https://bit.ly/fedexdropbox> to view FedEx drop box locations and pickup schedules. It's important to bring your sample to a drop box on the same day you collect it before the last Express pickup. Do not deliver sample to a drop box on Saturday or Sunday. Do not take to a FedEx office. Or refer to the flyer in your kit to schedule a FedEx pickup on the day of collection.

3




Wash and dry hands before opening the kit. Open your kit and place all the contents on a clean, dry surface.

4

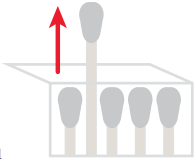


Take one of the cotton swabs out of its package. Do not touch the cotton tip of the swab with your hands. You will only need one of the cotton swabs but the rest have been provided as backups.

5

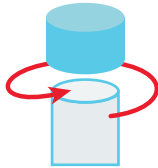


Screw off the top of the collection tube. Hold swab in one hand and collection tube in the other being careful not to spill the liquid. Do not drink the liquid.

6

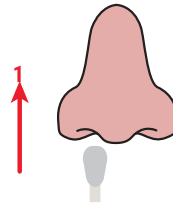


Insert the tip of the cotton swab into one nostril. The cotton swab does not need to be inserted far – insert just until the cotton tip of the swab is no longer visible. Rotate the swab in a circle around the entire inside edge of your nostril at least 3 times.

7

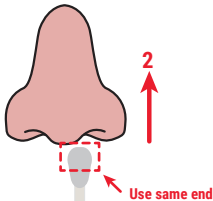

Take the cotton swab out of your nostril. Using the same end of the cotton swab, repeat step 6 in your other nostril.

8

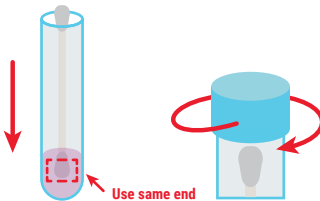


Remove the cotton swab from your second nostril and place in the collection tube. The end of the cotton swab that went into your nose should be placed into the tube first so that it sits down in the liquid. Screw the top of the collection tube back on.

9




Wash and dry hands thoroughly again. Insert collection tube into the biohazard specimen bag. Seal the biohazard specimen bag by closing the zip lock seal. Fold the specimen bag in half and lay the bag on one half of the gel pack.

10

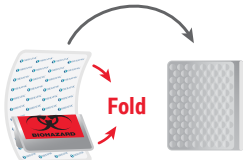

Fold the other half of the gel pack on top of the specimen bag. Place the specimen bag and gel pack into the insulated foil pouch. Remove the adhesive cover strip to seal the insulated foil pouch. Gel pack and specimen bag must be included in foil pouch for lab to generate a result.

11

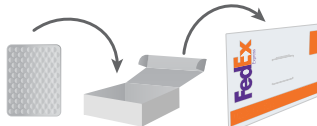

Place the insulated foil pouch into the shipping box and close the lid. Place the shipping box into the FedEx return pack. Remove the adhesive cover strip and seal the FedEx return pack closed.

12



Deliver the postage paid, pre-addressed FedEx return pack to a FedEx drop box. It's important to bring your sample to a drop box on the same day you collect it before last Express pickup. Do not deliver sample to a drop box on Saturday or Sunday. Do not take to a FedEx office. Or refer to the flyer in your kit to schedule a FedEx pickup on the day of collection.



How to Schedule a One-Time FedEx Pickup of your Pixel kit

Three Easy Steps

Use the three steps below to schedule your one-time pickup:

Step 1

Call FedEx at 1-800-463-3339. You will be asked a few questions.

FedEx: In a few words, please tell me what you're calling about.

Your answer: FedEx Express return pickup

FedEx: ...a pickup request for a return. To arrange a pickup for today or tomorrow, depending on availability, say "pickup."

Your answer: Pickup

FedEx: Is the word "stamp" written anywhere on your return label?

Your answer: No

FedEx: Please say or enter the tracking number (TRK#) on your return label

Your answer: Say or enter the tracking number

FedEx: What kind of return are you calling about – FedEx Ground, FedEx Home Delivery or FedEx Express?

Your answer: FedEx Express

Step 2

Provide additional information as requested:

- Pickup location address
- Number of packages to be picked up
- The day and time window for the pickup
- Is your shipment currently ready

Step 3

Note the confirmation and pickup number provided.

If you are transferred to FedEx customer service and a credit card or account number is requested, please state:

This is a FedEx Express return pickup

The FedEx Express return tracking number listed on the return label is XXXXXXXXXXXX (12 digits)



FROM Dewitt PluggHoe PluggHoe Ground Floor, 403A 1800 Peachtree Road Atlanta, GA 30361 US	(123) 456-7890	SHIP DATE: 19MAR19 ACTWGT: 1.00 LB CAD: 11895666@WWSX3300
TO: JOE SMITH FEDEX CORPORATION SAMPLE LABEL 3650 HACKS CROSS RD MEMPHIS TN 38125 (901) 222-2222 INV PC		BILL SENDER
REF: SAMPLE PRINT LABEL RETURN		DEPT:
		
→ TRK# XXXXXX 8391 12 Digit Return Tracking Number		RETURNS MON-FRI PRIORITY OVERNIGHT
		38125 TN-US
		



IMPORTANT INFORMATION!



BEFORE YOU COLLECT YOUR SAMPLE:

Is it a week day (Monday-Friday)?

Have you confirmed FedEx drop box pickup times or scheduled a home pickup on a weekday? See the Included FedEx flyer.

Will your sample be picked up on the SAME DAY as you collect it?



BEFORE YOU COLLECT YOUR SAMPLE:

Have you registered your kit in your Pixel by LabCorpTM account?

Have you placed BOTH the gel pack AND your sample in the foil pouch?



YOUR SAMPLE MAY BE REJECTED IF YOU:

Fail to include BOTH sample AND gel pack in the foil pouch.

Drop off in a FedEx drop box on a Saturday or Sunday, or after the last pickup of the day.

Schedule a FedEx pick up on a Saturday.

Try to drop your sample at a FedEx office location, rather than a drop box.