

How to burn a DVD for Breast Studies

1. Send the images to EWS-MRI workstation
2. On the EWS-MRI workstation, find the study in the "workstudy list" window.
3. Click to select the exam
4. In the lower window, click again to select the exam
5. In the window below that, choose the desired sequences by using CTRL+CLICK
6. With these options selected, right click and choose "copy"
7. Choose the option that says something like "Recordable Drive F"
8. Leave all of the checkboxes below BLANK except for the "Dicom Viewer". If you DO NOT want the Dicom Viewer information burned for your images, UNCHECK this box.
9. Hit "apply" or "ok" or whatever button there is to allow you to burn the disk
10. The disk will start to burn.
11. Leave the computer alone for a while. This takes several minutes.