

OHSU MRI PROTOCOLS, DOCUMENTS, PROCEDURES AND OTHER INFORMATION FOR TECHS

Revised 1/25/2011

ADULT MSK PROTOCOL SI JOINTS W/WO

MSK	SI JOINTS W/WO CONTRAST						XL TORSO COIL	
	LOCALIZER	MODE:	SLICE	GAP	FAT SAT	FOV	SCAN RANGE:	
	REF SCAN							
AXIAL OBL	T1	TSE	3 mm	1 mm	None	25 cm	Angle to Sacrum	
AXIAL OBL	Mid TE T2 Fat Sat	TSE	3 mm	1 mm	SPAIR	25 cm	Angle to Sacrum	
OBL COR	T1	TSE	3 mm	1 mm	None	25 cm	Angle Parallel to Sacrum	
OBL COR	Mid TE T2 Fat Sat	TSE	3 mm	1 mm	SPAIR	25 cm	Angle Parallel to Sacrum	
CONTRAST INJECTION								
OBL COR	T1 Fat Sat	TSE	3 mm	1 mm	SPIR	25 cm	Angle Parallel to Sacrum	
AXIAL OBL	T1 Fat Sat	TSE	3 mm	1 mm	SPIR	25 cm	Angle to Sacrum	
NOTES:	<i>USE REST SLAB PROXIMAL TO REDUCE FLOW AND PULSITILE MOTION</i>							